


RealisticVue™

HERA W10 Quick Guide



1. Probe and Preset

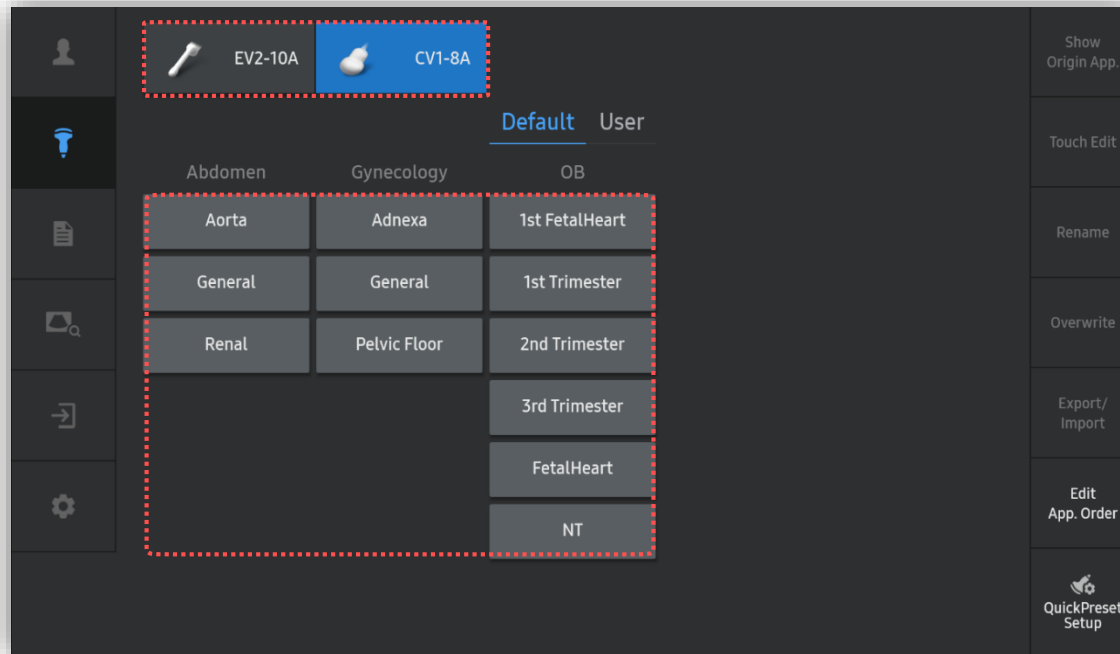

※ RealisticVue™ can be operated under the following conditions :

Probe	Application
CV1-8A EV2-10A	All presets

2. Volume acquisition




① Volume acquisition

Acquired volume data with the structure you wish to apply RealisticVue™.


★Tips

For fetal face 3D evaluation, obtain fetus profile as much as clear without interruption of hand or cord.

3. Apply RealisticVue™

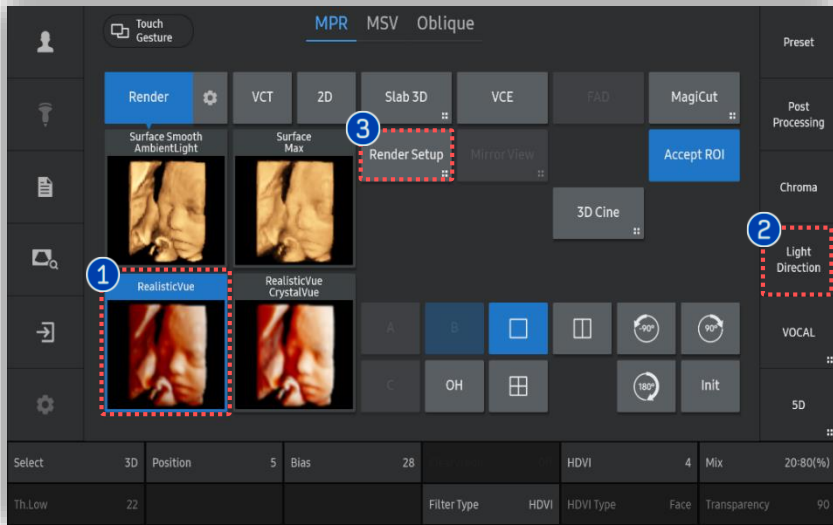


- | | | |
|---|-----------------|---|
| 1 | RealisticVue™ | Tap [RealisticVue™] to apply it. |
| 2 | Light Direction | Tap [Light Direction] within short cut keys when you adjust the direction of light freely by trackball. |
| 3 | Render Setup | Tap [Render Setup] on the touch screen to adjust parameters of RealisticVue™. |

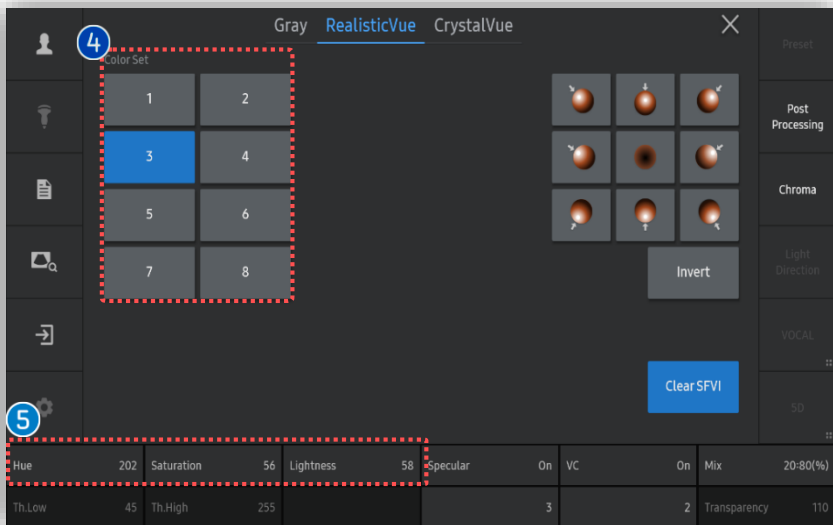

- | | | |
|---|------------------|--|
| 4 | Color Set | Select color of RealisticVue effect among 8 designated options. |
| 5 | Color parameters | For fine control of color effect, adjust Hue, Saturation and Lightness. <ul style="list-style-type: none"> ▪ Hue: Shade of a color ▪ Saturation: Purity or intensity of color ▪ Lightness: Brightness of color |

3. Apply RealisticVue™

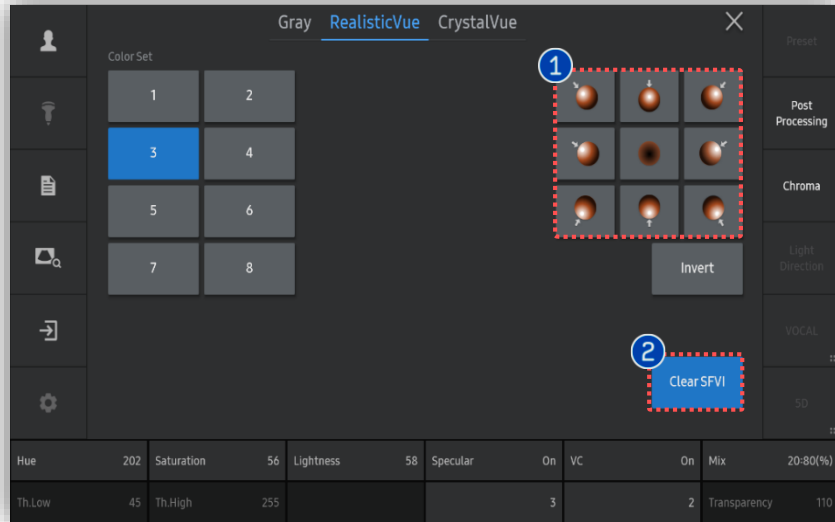

1 Pre-Defined Light Direction

Tap preferred light direction among 9 designated options.

2 Clear SFVI

Smart Filter Volume Imaging. It reduces noise.

- The features, options may not be commercially available in some countries.
- Sales and shipments are effective only after the approval by the regulatory affairs. Please contact your local sales representative for further details.
- This Quick guide does not include all of the details of instruction, for more detail, please refer to HERA W10 User Manual.
- Do not distribute this document to customers unless relevant regulatory and legal affairs officers approve such distribution.
- This User Quick Guide is based on HERA W10 V1.03.
- Disclaimer: Some Images in this content were obtained from other system.

SAMSUNG MEDISON CO., LTD.

© 2024 Samsung Medison All Rights Reserved.

Samsung Medison reserves the right to modify the design, packaging, specifications, and features shown herein, without prior notice or obligation.