


## CEUS+

### RS85 Prestige Quick Guide



# 1. Start CEUS+

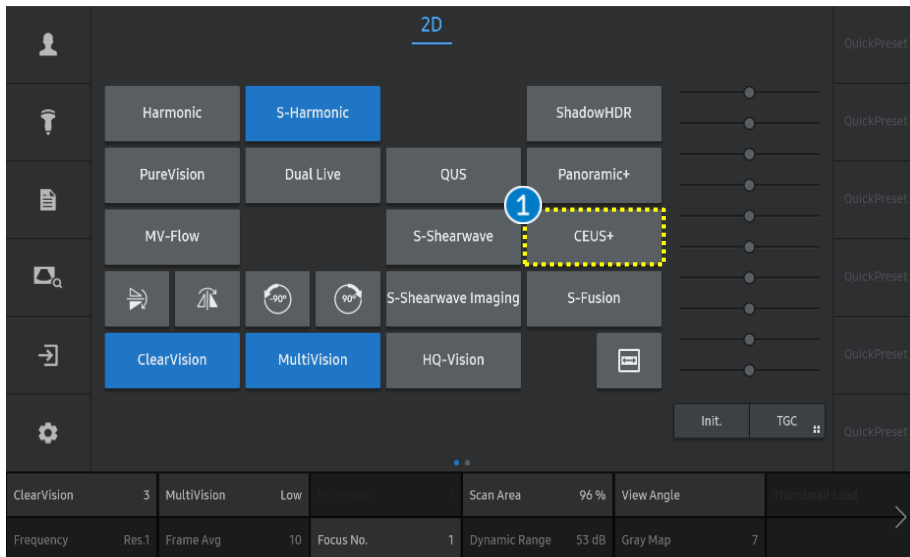

1 CEUS+

Tap the [CEUS+] on the touch screen.

When entering the [CEUS+] mode, Dual mode is activated as a default.




1 CEUS+ mode

Contrast Enhanced Ultrasound image will be displayed on the left screen.

2 2D mode

2D image will be displayed on the right screen.

## 2. Select a Agent Type and Timer

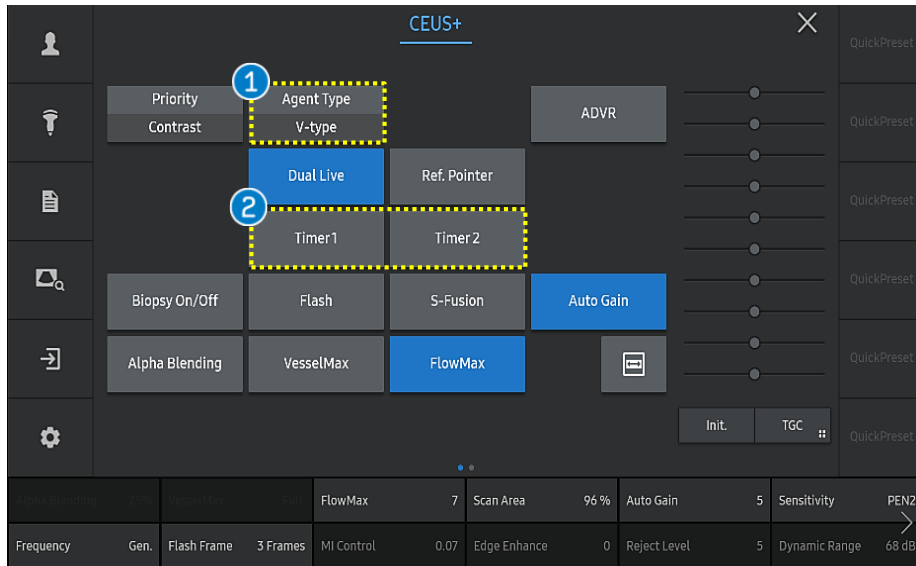


### 1 Agent Type

Tap the [Agent Type] button on the touch screen.


<b>V-Type</b>	A contrast agent used in a <b>low MI</b> environment. (approximately below 0.1)
<b>Z-Type</b>	A contrast agent used in a <b>medium MI</b> environment. (approximately above 0.1).

### 2 Timer

Tap the [Timer1] or [Timer2] button on the touch screen to start the timer. When touching the button again, the timer is reset.

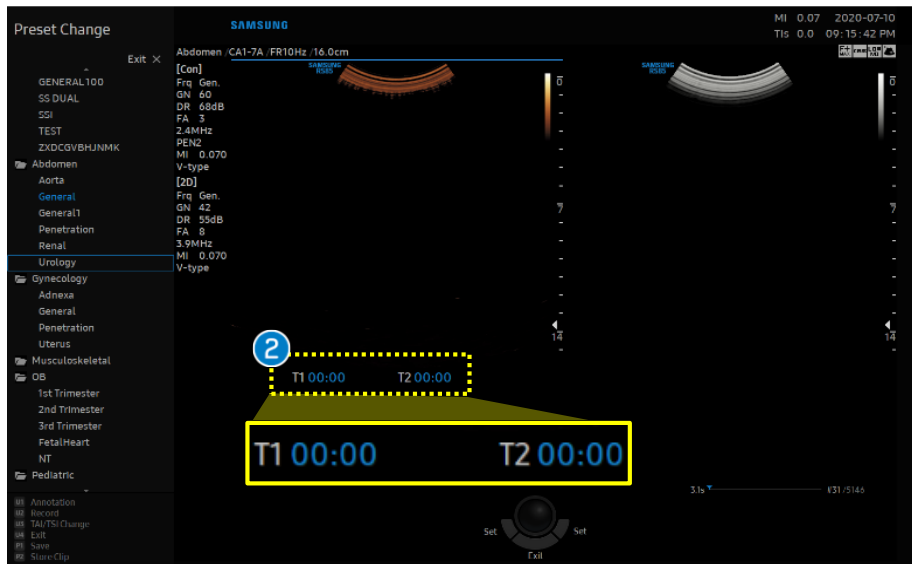

# 3. Adjust parameters (1)

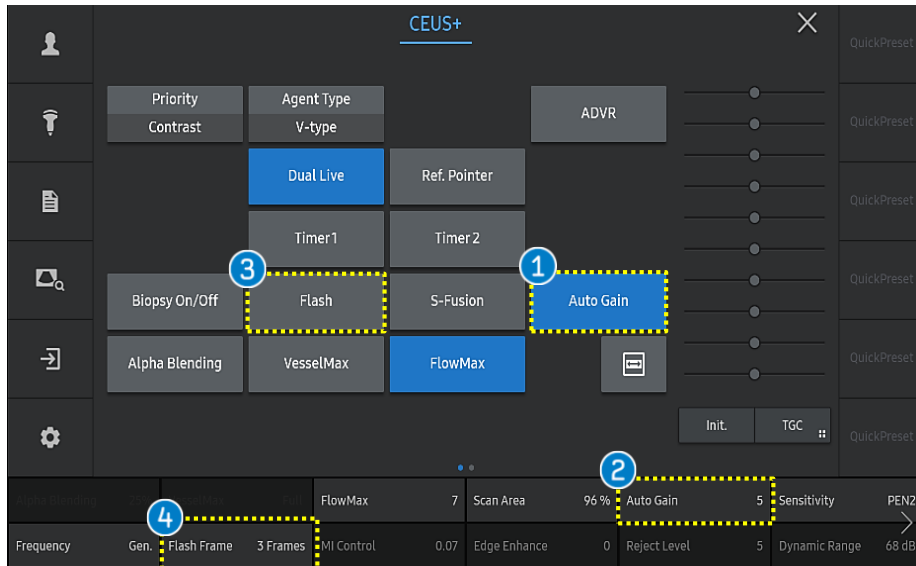

## Notes

Auto Gain is available depends on the probe and preset.

### 1 Auto Gain

It suppresses the saturated signals and provides proper brightness with automatic brightness control.

### 2 Auto Gain (Offset)

It adjusts the offset of auto gain.

- Low level: Image is getting darker with heavy suppression on bright signals.
- High level: Image is getting brighter with mild suppression on bright signals.


### 3 Flash

Bursts the micro-bubbles of contrast agent within the scan area by generating a high MI pulse.

### 4 Flash Frame

Select a flash duration. It bursts the bubbles for a selected frame(duration).

# 3. Adjust parameters (2)

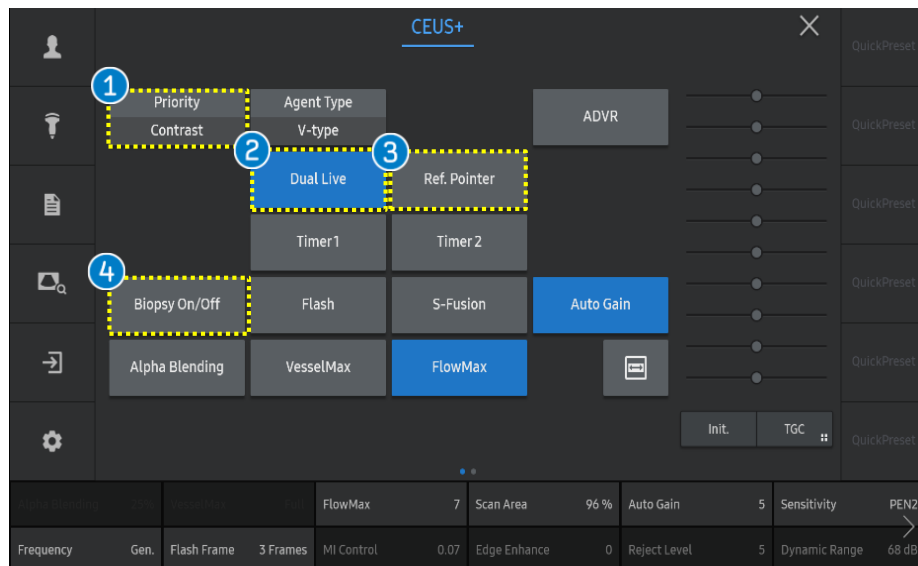

## 1 Priority

Tap the [Priority] button on the touch screen to adjust parameters of the selected Priority setting.

- Contrast: CEUS image
- 2D: 2D image

## 2 Dual Live

Tap the [Dual Live] button to select image format dual or single.

- Dual: Displays the 2D and contrast enhanced US image.
- Single: Displays the CEUS or 2D as full screen depends on the priority.


## 3 Ref. Pointer

Ref. Pointer can be used as a synchronizing tool in Dual Live mode. It can help to check the location of the lesion on both windows.

## 4 Biopsy On/Off

If you want to display the biopsy guideline, tap the [Biopsy On/Off] button.

# 3. Adjust parameters (3)

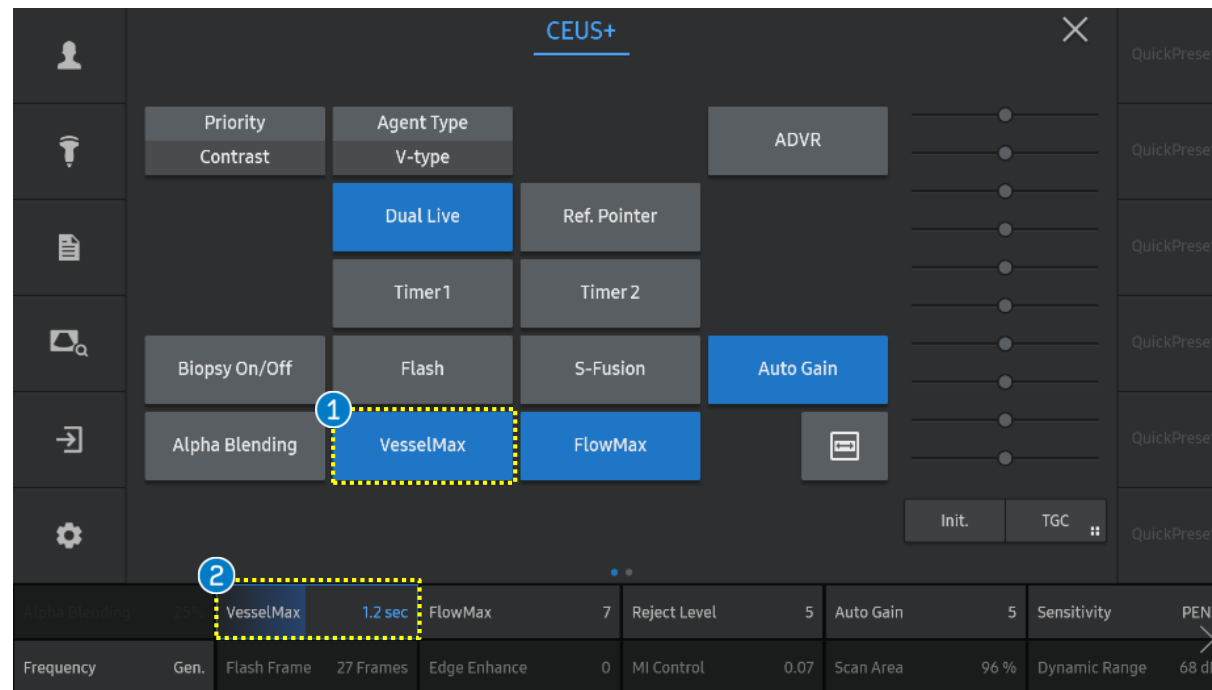

## 1 VesselMax

Tap the [VesselMax] button to activate it.

It accumulates contrast image frames over time to provide a visualization of small vessels clearly.

## 2 VesselMax (Time duration)


Available to adjust the time duration

- Low level: Accumulates the frames less.
- High level: Accumulates the frames more.

### Notes

VesselMax can be adjusted on freeze and unfreeze(active mode) status both.

## 3. Adjust parameters (4)

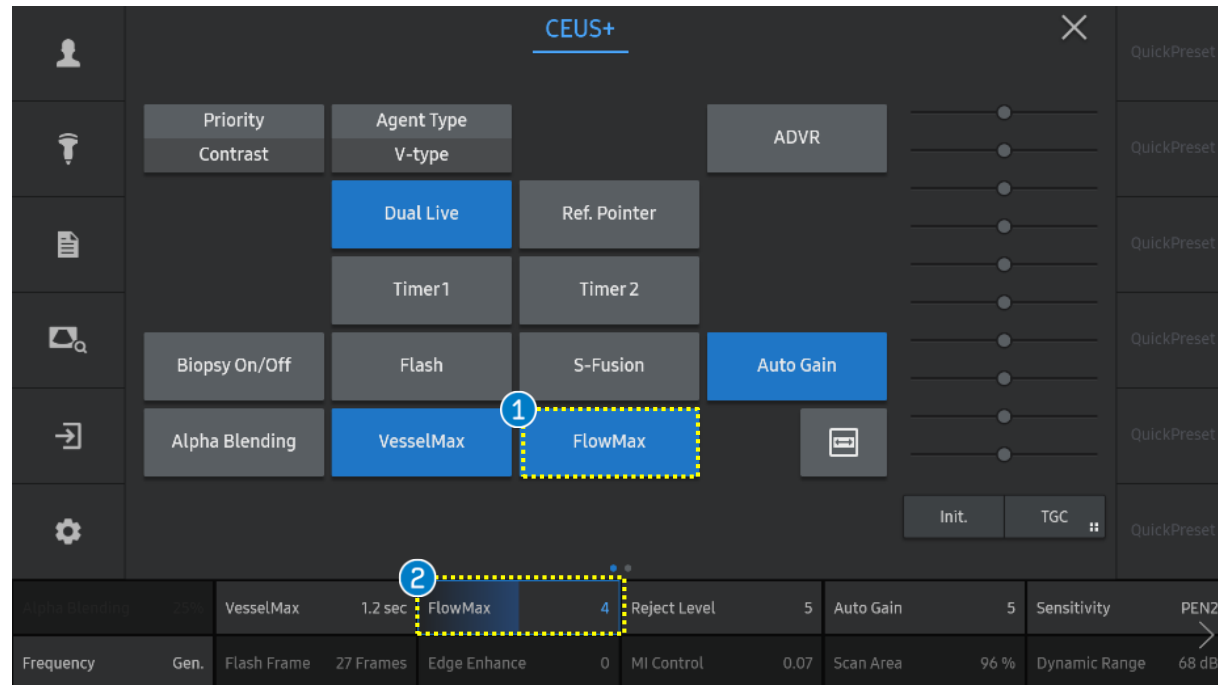

### 1 FlowMax

Tap the [FlowMax] button to activate it .

It suppresses the saturated signal which comes from the tissue, not the contrast agent.

### 2 FlowMax (Index)


Available to adjust the FlowMax index

- Low level: Suppresses the tissue signal less.
- High level: Suppresses the tissue signal more.

### Notes

FlowMax can be adjusted on freeze and unfreeze(active mode) status both.

# 3. Adjust parameters (5)

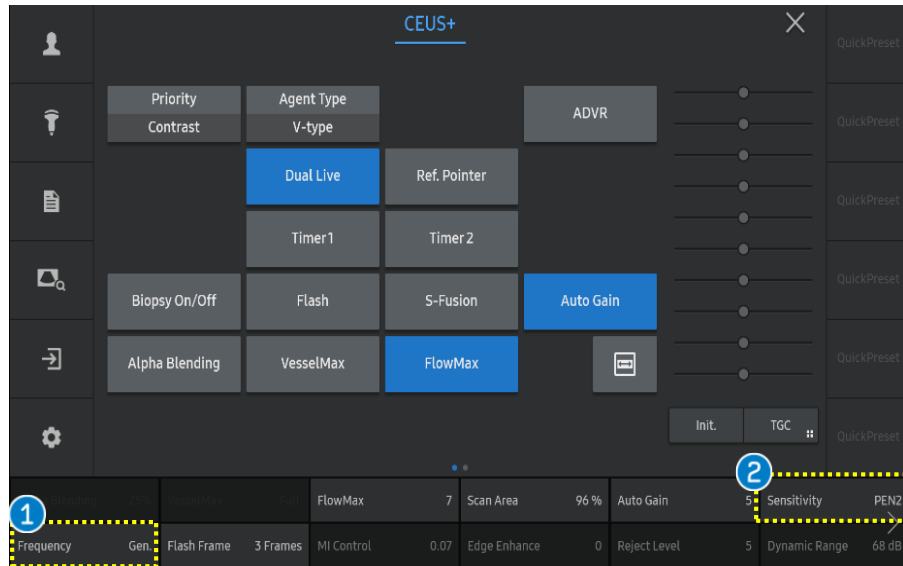

## 1 Frequency

Select the [Frequency] for CEUS.

Available Frequencies are variable depending on the selected Sensitivity.

## 2 Sensitivity

Select the [Sensitivity] of the contrast agent as GEN, PEN1, PEN2.


<b>GEN</b>	Good resolution with fine pixel size.
<b>PEN1</b>	High sensitivity with good penetration.
<b>PEN2</b>	Well balanced setting between sensitivity and resolution.

### Notes

Sensitivity parameter is activated in particular probe and preset.



## 3. Adjust parameters (6)

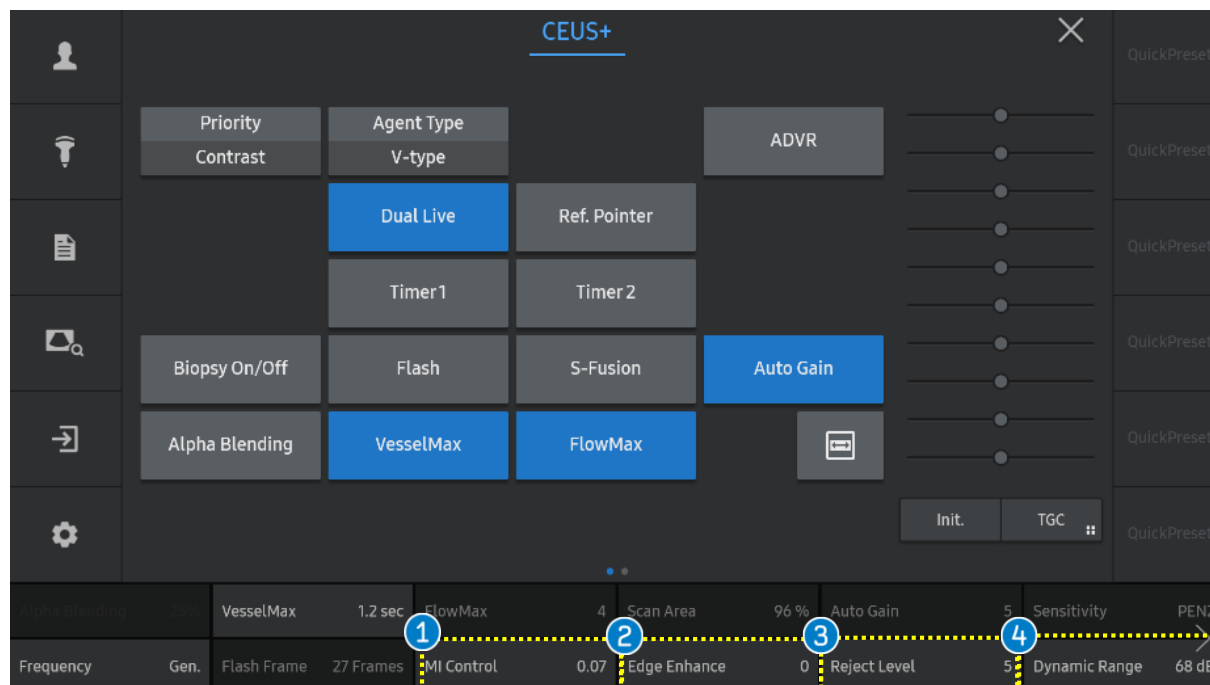

### 1 MI Control

Defines the Mechanical Index of the Ultrasound beam.

### 2 Edge Enhance

Enhances the edge of image on contrast mode or 2D mode (depends on the selected Priority).


### 3 Reject Level

- Removes low signals on contrast mode or 2D mode (depends on the selected Priority).
- As reject level is increased both tissue and contrast signals will be rejected.

### 4 Dynamic Range

Adjusts the dynamic range on contrast mode or 2D mode (depends on the selected Priority).

# 3. Adjust parameters (7)

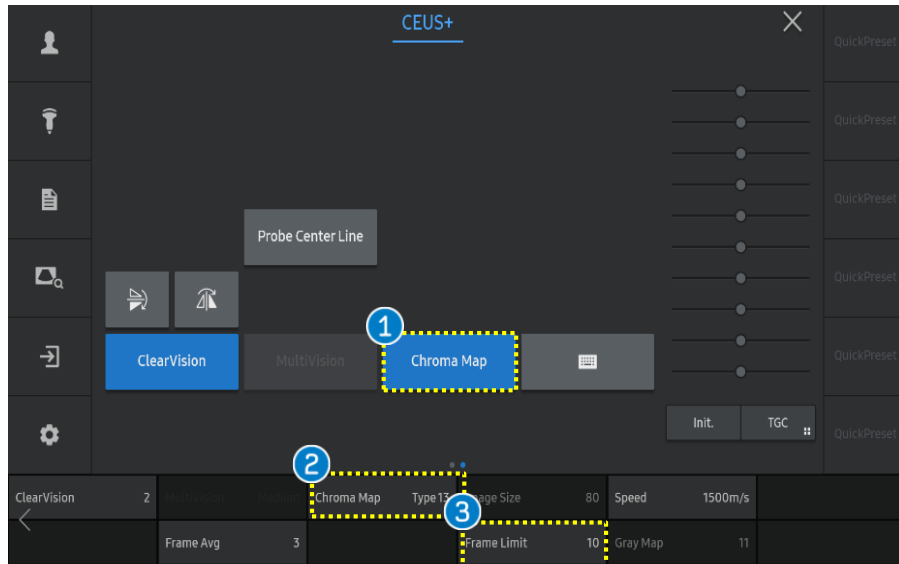

## 1 Chroma map

Tap the [Chroma Map] button to adjust the color-map of the CEUS image or 2D image (depends on selected Priority).



## 2 Chroma map type

Select a type of Chroma map for your preference.

## 3 Frame Limit

The [Frame Limit] controls frame rate. Select a frame limit from 2 to the maximum frame limit for the current scan mode. Maximum frame limit depends on the current imaging parameter.

- Low level: Set the low frame rate.
- High level: Set the high frame rate.

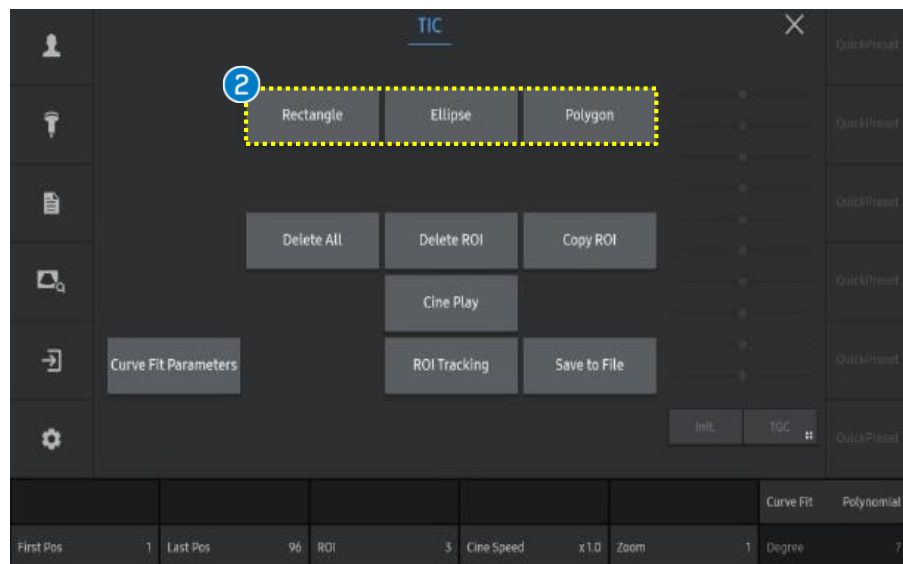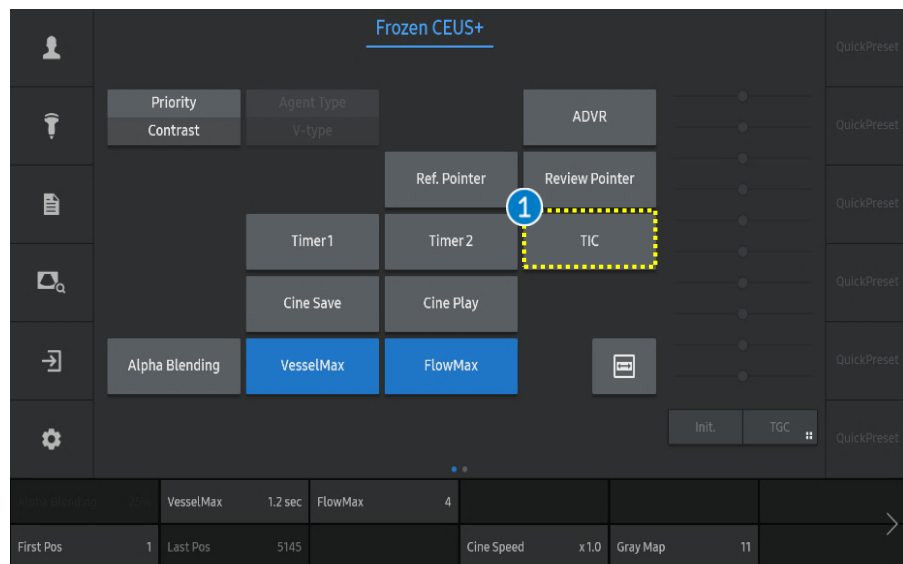

## 1 TIC

Tap the [TIC] button on the touch screen in the freeze status.

- TIC (Time Intensity Curve) is a curve that represents the change of the average intensity within ROI over time.

Select the ROI type on the touch screen.

- **Rectangle and Ellipse**


Press the [Set] button to put the start point of ROI. Then adjust the size of ROI using trackball. Press the [Set] button again to complete ROI.

- **Polygon**

Press the [Set] button on the start point and every angular point of the lesion. To fix the ROI, press the [Exit] button on the control panel.


## 2 ROI Type

## 4. Time Intensity Curve (2)

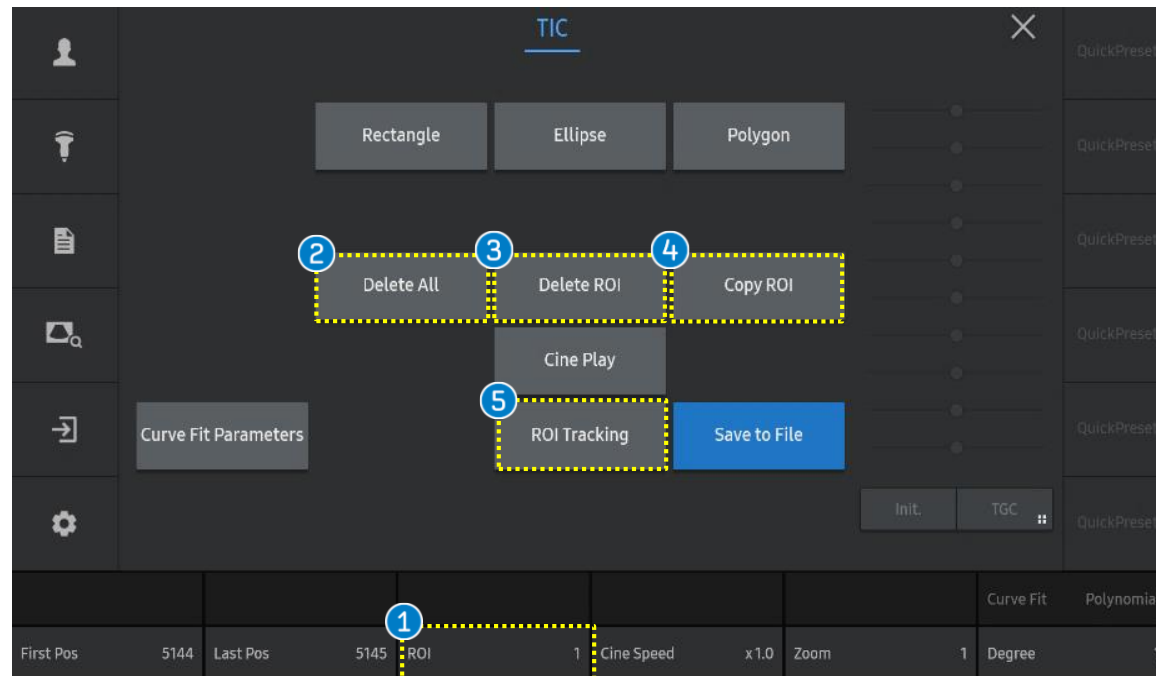

- |                |   |
|----------------|---|
| ① ROI          | To copy or delete ROI, select the ROI index.                            |
| ② Delete All   | Tap the [Delete All] to delete all ROIs.                                |
| ③ Delete ROI   | Select the ROI and tap the [Delete ROI].                                |
| ④ Copy ROI     | Select the ROI and tap the [Copy ROI] to copy the selected ROI.         |
| ⑤ ROI Tracking | Updates the ROI location automatically by tracking the tissue movement. |


# 4. Time Intensity Curve (3)

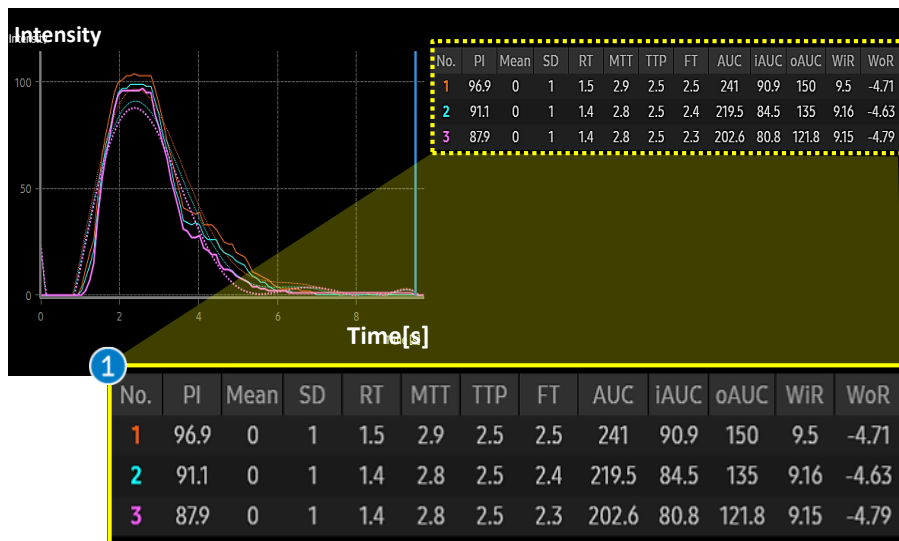



① TIC Parameter

The TIC parameters are displayed differently depending on curve fit you selected.

<b>PI</b>	Peak Intensity
<b>Mean</b>	Mean Intensity
<b>SD</b>	Standard deviation
<b>RT</b>	Rise Time
<b>MTT</b>	Mean Transit Time
<b>TTP</b>	Time To Peak
<b>FT</b>	Fall Time
<b>AUC</b>	Area Under the Curve
<b>iAUC</b>	Wash in Area Under the Curve
<b>oAUC</b>	Wash out Area Under the Curve
<b>WiR</b>	Wash in Rate (maximum slope)
<b>WoR</b>	Wash out Rate (minimum slope)

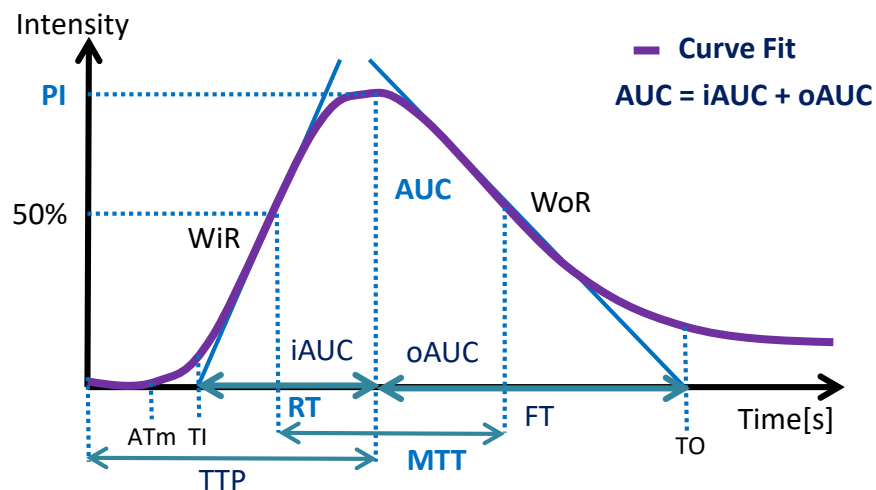

# 4. Time Intensity Curve (4)

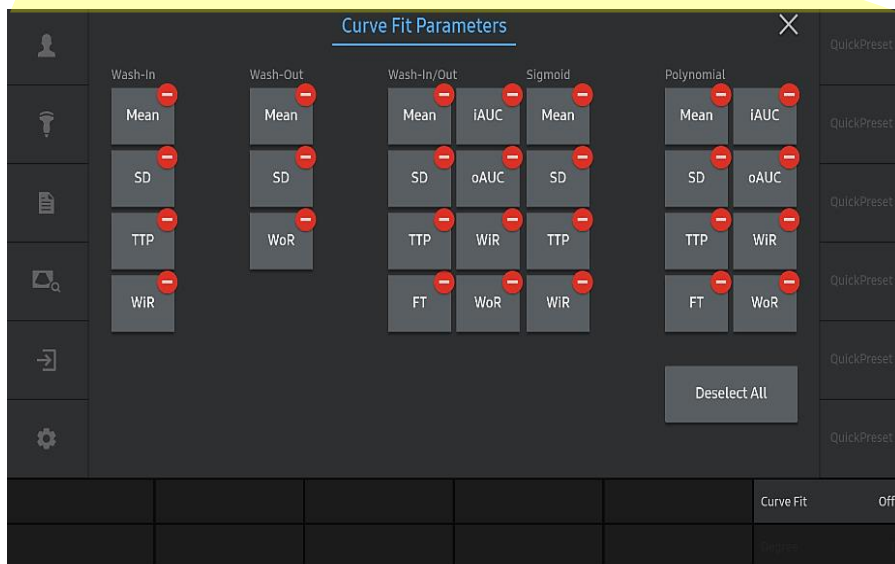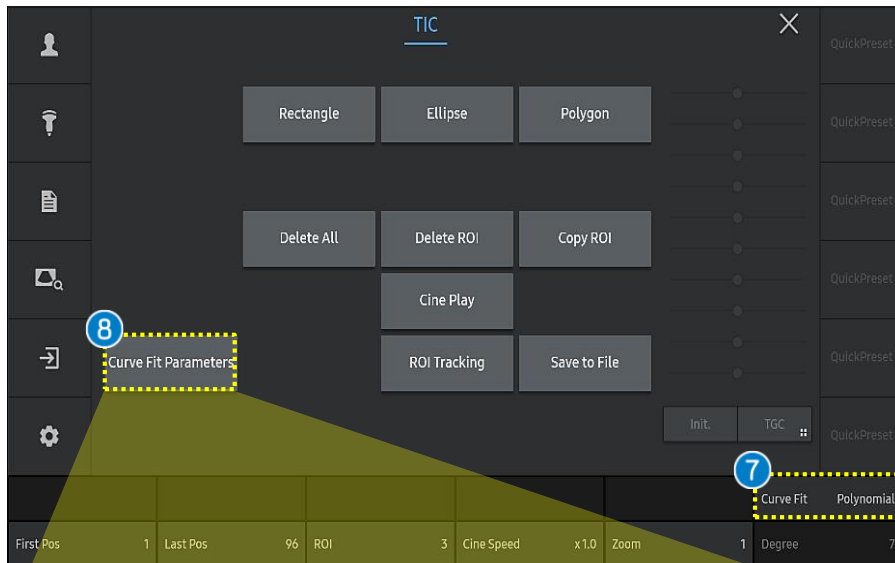

## 7 Curve Fit


Provides suitable curve shape to measure blood flow parameters accurately and consistently.

Wash-in	Fit the TIC as an increasing curve.
Wash-out	Fit the TIC as a decreasing curve.
Wash-in/out	Fit the TIC as a curve that shows both increase and decrease.
Sigmoid	Fit the TIC in a S-shaped curve.
Polynomial	Suitable for local wash-in and wash-out rate estimation during bolus injection. You can adjust the degree of polynomial from 1 to 10. As the degree increases, curve will be closer to TIC.

## 8 Curve Fit Parameters

Select the TIC parameters to display on the screen.

# 4. Time Intensity Curve (5)

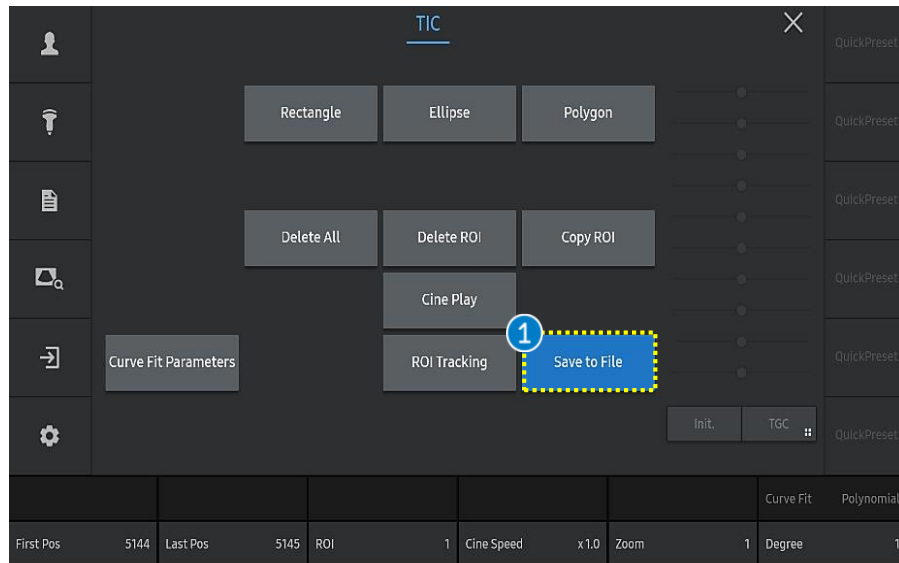

## 1 Save to File


Insert an external memory device into the system before saving to file. Tap the [Save to File] button. TIC is saved in csv file.

## 2 Directory & Name

Choose directory and name of the file.

## 3 Save

Click the [Save] button to complete export.

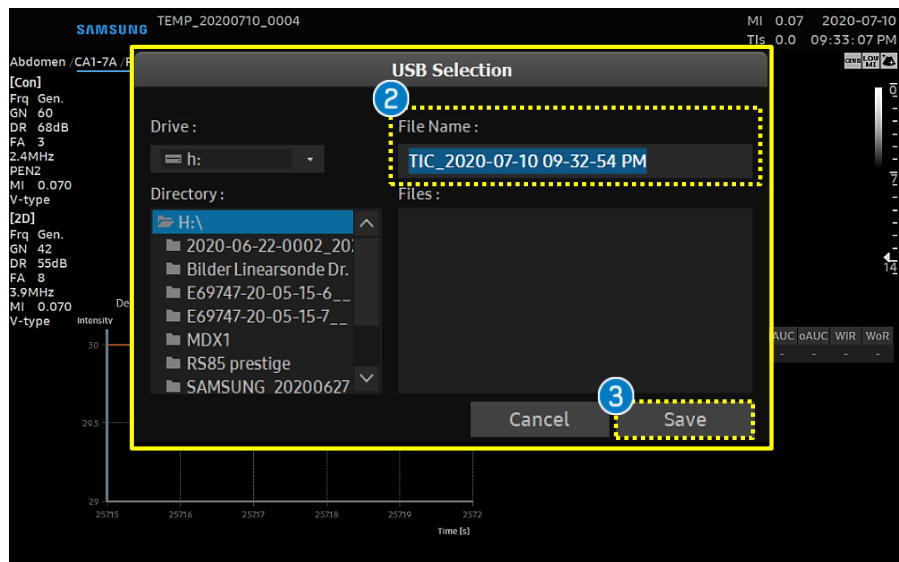

**SAMSUNG**



- The features, options may not be commercially available in some countries.
- Sales and shipments are effective only after the approval by the regulatory affairs. Please contact your local sales representative for further details.
- This Quick guide does not include all of the details of instruction, for more detail, please refer to RS85 User Manual.
- Do not distribute this document to customers unless relevant regulatory and legal affairs officers approve such distribution.
- This User Quick Guide is based on RS85 V2.08.
- Disclaimer: Some Images in this content were obtained from other system.

**SAMSUNG MEDISON CO., LTD.**

© 2024 Samsung Medison All Rights Reserved.

Samsung Medison reserves the right to modify the design, packaging, specifications, and features shown herein, without prior notice or obligation.