### SAMSUNG





Produced by Clinical Training Center / SAMSUNG MEDISON CO.,LTD


# Panoramic+

## V series Quick Guide

#### **V** series

Panoramic+

### 1. Start Panoramic

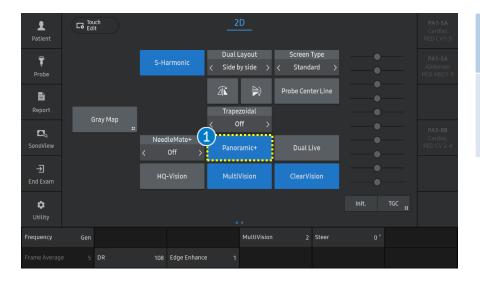


2 Patient	Lo Touch Edit	Panoramic+ Ready	PA1-5A Cardiac PED CV1-5
<b>Ţ</b> Probe		Screen Type < Standard >	PA1-5A Abdomen PED ABD1-5
Report		Width Scale ::	
<b>D</b> a SonoView		_	PA3-88 Cardiac PED CV 2-4
- <b>권</b> End Exam		2 Start	
Ctility			Init. TGC :
			Thumbnail
			Thumbnail Load


1 Panoramia	To acquire a panoramic image, select [Panoramic+] button on the touch screen.	
2 Start	When you are ready to acquire the image, press [Start] button or [Set] button.	

**V** series

#### Panoramic+

### 2. Acquire Panoramic image

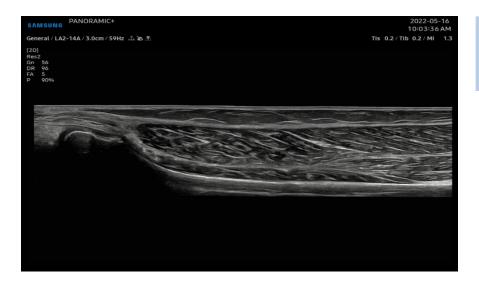

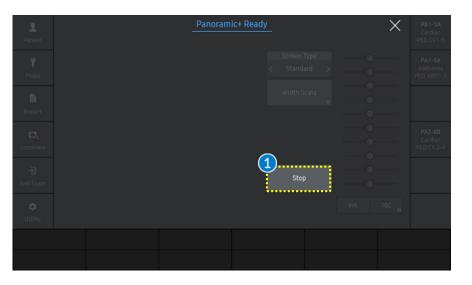



Stop

To complete acquisition of image, Press [Stop] button or [Set] button. **V** series

Panoramic+

#### 3. Parameters

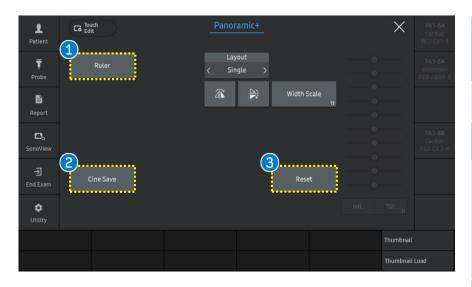


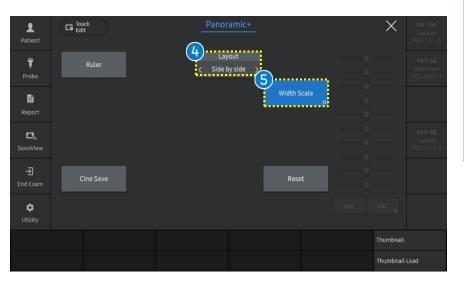

1	Ruler	When selected, ruler is displayed on the panoramic image.
2	Cine Save	Save Cine images.
3	Reset	Returns to the panoramic ready screen.
4	Layout	It specifies how the panoramic image is displayed on the screen. You can select the layout type among [Single], [Side by side], and [Up/Down].
5	Width Scale	Image size of left window will be changed according to width scale setting in Side by side layout. Among [Single], [Side by side] and [Up/Down], only [Side by side] layout can be used in Width Scale.


#### Panoramic+

#### 4. Width Scale

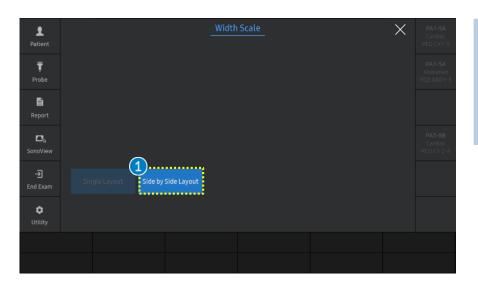

Turn On

If you select [Side by Side Layout] in Width scale, the 2D image will display the ROI as a full image. Move the ROI with the trackball to display the desired 2D image area.

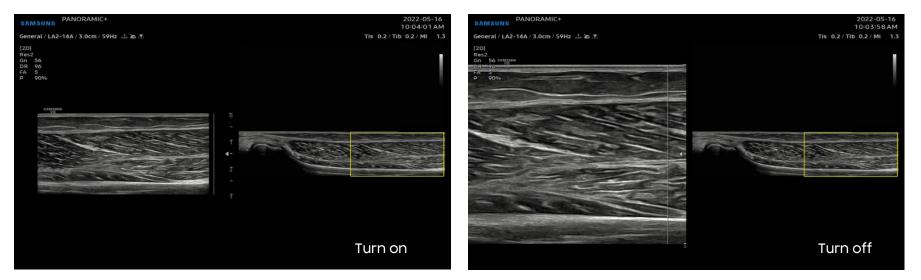

- The features, options may not be commercially available in some countries.
- Sales and shipments are effective only after the approval by the regulatory affairs. Please contact your local sales representative for further details.
- This Quick guide does not include all of the details of instruction, fore more detail, please refer to V series User Manual.
- Do not distribute this document to customers unless relevant regulatory and legal affairs officers approve such distribution.
- This User Quick Guide is based on V series V1.05.
- Disclaimer: Some Images in this content were obtained from other system.

#### SAMSUNG MEDISON CO., LTD.

© 2024 Samsung Medison All Rights Reserved. Samsung Medison reserves the right to modify the design, packaging, specifications, and features shown herein, without prior notice or obligation.